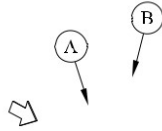
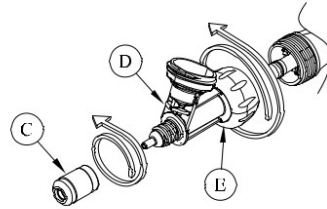
## CLEANING INSTRUCTIONS

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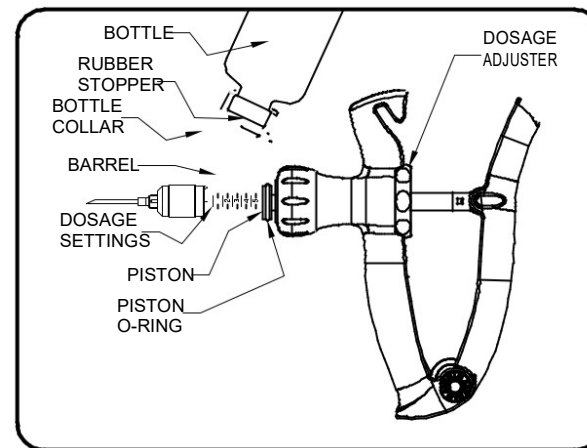
## TO ATTACH BOTTLE USING BOTTLE COLLARS:

\* Some units are equipped with spike collars. There are 3 sizes of spike collars. There is a collar for 100ml bottles (20mm), another for 250ml bottles (30mm), and a third for bottles with a 33mm diameter cap. Vaccinators may be supplied with any combination of the three. However, operation instructions are the same for each.

- 1.) Completely unscrew the spike collar from the barrel and remove spike cover.
- 2.) Insert the bottle neck into the "T"-slot cut-out area of the spike collar.
- 3.) Hold bottle with assembled spike collar in one hand, while pushing the draw spike through the hole already pierced in the rubber stopper.
- 4.) Hold firmly to vaccinator & screw-on the spike collar.

## TO PRIME VACCINATOR:

- 1.) Remove protective cover from needle.
- 2.) Align front of piston with the line of the largest dose setting. \*NOTE: SOME VACCINATORS ARE FIXED-DOSE ONLY. THIS MEANS THAT THE DOSE SETTING CANNOT BE ADJUSTED (EVEN THOUGH THE ADJUSTER WILL TURN). ADJUSTABLE DOSE VACCINATORS WILL HAVE GRADUATED DOSAGE SETTINGS PRINTED ON THE BARREL AS SHOWN. FIXED, NON-ADJUSTABLE VACCINATORS WILL HAVE BARRELS PRINTED WITH ONE DOSE SETTING ONLY, WHICH REPRESENTS THE DOSAGE TO WHICH THE VACCINATOR IS FIXED. IF YOU ARE USING A FIXED-DOSE VACCINATOR SKIP THIS STEP AND GO TO STEP #3.
- 3.) Tip vaccinator up at a slight angle, with the needle pointing up, ensuring that the draw spike is below the uid level inside the bottle. Bottles that are low on vaccine, may allow the spike out of the vaccine when tipped and allow the spike to draw air from the bottle into the barrel of the vaccinator.
- 4.) Squeeze vaccinator handle, then release quickly to draw uid into the barrel.
- 5.) Remove any air bubbles from the barrel by repeating step 4 being careful not to waste uid.

## TO SET DOSAGE:

Align front of piston with the line of the largest dose setting using the dose adjuster. \*NOTE: SOME VACCINATORS ARE FIXED-DOSE ONLY. THIS MEANS THAT THE DOSE SETTING CANNOT BE ADJUSTED (EVEN THOUGH THE ADJUSTER WILL TURN). ADJUSTABLE DOSE VACCINATORS WILL HAVE GRADUATED DOSAGE SETTINGS PRINTED ON THE BARREL AS SHOWN. FIXED, NON-ADJUSTABLE VACCINATORS WILL HAVE BARRELS PRINTED WITH ONE DOSE SETTING ONLY, WHICH REPRESENTS THE DOSAGE TO WHICH THE VACCINATOR IS FIXED.

## LUBRICATION:

If vaccinator becomes sluggish, clean thoroughly using the cleaning instructions on back. You may not need to disassemble the spike assembly. However, you will need to remove the barrel assembly, clean and apply a couple of drops of Prima Tech lubricant to the piston o-ring. Many times, product failure is due to insufficient cleaning (dried vaccine in valves, bung material caught in draw spike, etc.). For complete cleaning instructions, please see back panel.